
PANTALLA FACIAL
FACE SHIELD
ÉCRAN FACIAL

325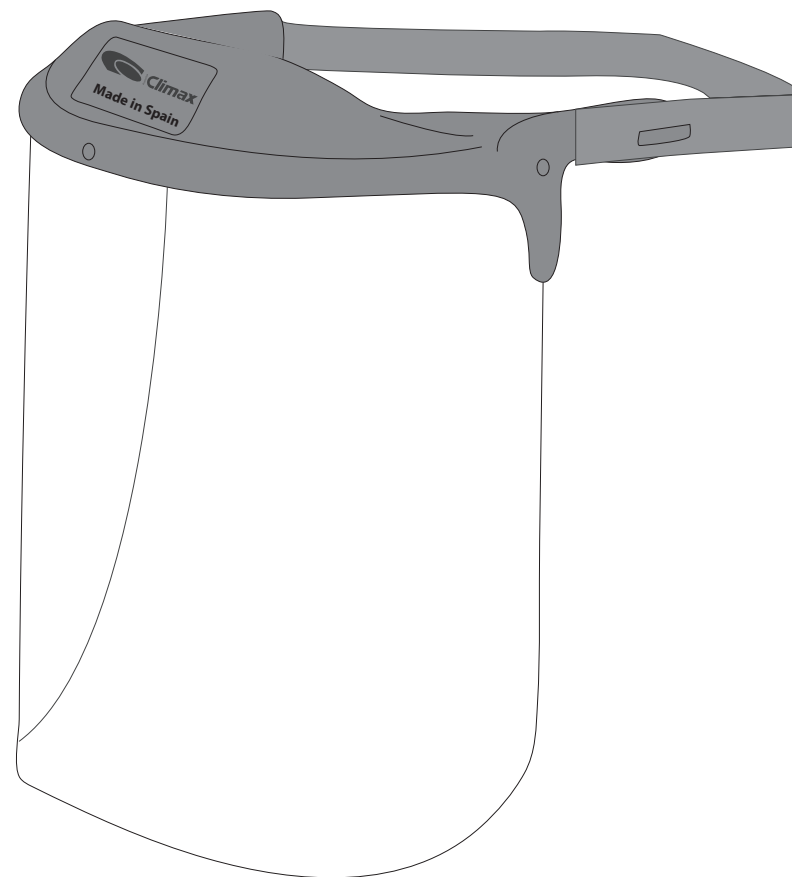

Made in Spain

www.productosclimax.com

PANTALLA FACIAL 325

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FACE SCREEN 325

This product has been manufactured in accordance with the requirements of Regulation (UE) 2016/325, for basic use, following the Recommendation for Use RFU PPE-R/03.031 Version 1 based on Recommendation (EU) 2020/403 of the Commission, of 13 March 2020 on conformity assessment and market surveillance procedures within the context of the COVID-19 threat, to protect against droplets and splashes, as indicated in certificate No. 0161/xxxx/yy of AITEX, Plaza Emilio Sala nº 1, Alcoi, España, Notified Body 0161.

This PPE contains the following materials:

Frame: ABS, polyether foam

Visor: Polycarbonate

Fastening system. Polyester

APPLICATION

For the protection of eyes and face in activities requiring protection from droplets and splashes only during the COVID-19 emergency. This face screen has been manufactured exclusively for protection against droplets and splashes during the COVID-19 emergency and is not destined for general use, nor should it be used for other purposes.

USE

The Face Screen 325 is placed on the user's head, adjusting the harness until it fits perfectly, leaving "hands free" for carrying out operations.

CLEANING RECOMMENDATIONS

The equipment can be cleaned after each use by impregnating its entire surface, including the fastening band, with 70° - 90° ethyl alcohol

MAINTENANCE

To protect the visor, follow these storage instructions.

STORAGE

The visor has a protective film on both sides, which protects it from dust and damp.

Keep it in its protective bag when not in use.

Store at a temperature between 5 and 40 °C, with humidity below 90%, and in a clean place, away from sources of heat.

PACKAGING

Supplied in a cardboard box containing 10 units

MARKINGS AND PERFORMANCE:

The PPE 325 should be marked:

RFU 03.031/V1-Climax -1-3

This marking refers to:

RFU 03.031/V1: that it has been certified according to Recommendation for use RFU PPE-R/03.031 Version 1 based on Recommendation (EU) 2020/403 of the Commission, of 13 March 2020 conformity assessment and market surveillance procedures within the context of the COVID-19 threat.

Climax: Identification of the manufacturer

1: Optical class 1

3: Protection against liquid droplets and splashes

USE LIMITATIONS

This face screen has been manufactured exclusively for protection against droplets and splashes during the COVID-19 emergency and is not destined for general use, nor should it be used for other purposes. This PPE offers no protection against aerosols or particles in suspension.

ASSEMBLY INSTRUCTION, APPROPRIATE ACCESSORIES AND SPARE PARTS.

Only use accessories made by Productos Climax. The face screen has the following spare parts:

- Clear visor

- Fastening elastic band

To replace the visor, press lightly outwards. Fit the new visor, centring it in the deflector (black piece), and press

lightly until the holes in the visor are filled with the projections on the frame.

WARNING:

The materials that come into contact with the user's skin could cause allergies in sensitive people.

Replace the PPE if it is seen to be damaged in any way.

It is recommended to replace the visor if it is scratched or damaged.

The face screen model 325 does not offer protection against mechanical impacts.

The face screen model 325 should only be used in the health sector, emergency services as well as activities requiring protection against droplets and splashes and only during the current COVID-19 emergency.

The face screen model 325 protects against liquid droplets and splashes.

This protection can be combined with surgical masks, filter masks, half masks or full masks.

The PPE has a service life of 5 years.

This service life will also depend on how it is used, maintenance, storage, etc.

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